# GELITA-SPON® RAPID<sup>3</sup>

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EN

Dry Use Absorbable Gelatin Sponge Hemostat (USP)

1. Product Description  
The GELITA-SPON® RAPID Dry Use Absorbable Gelatin Sponge Hemostat is made of 100% purified gelatin. It is intended for use as topical hemostat. The product's activated capillary function provides hydrophilic properties allowing it to absorb fluid rapidly upon contact. It is insoluble in water and fully absorbable. Its porous structure and interstices enables the product to absorb more than 40 times its own weight of blood and fluids. The absorbative capacity of GELITA-SPON® RAPID is a function of its physical size, increasing as the amount of material increases. In addition, because of its neutral character and nature, it may be used as a carrier for any drug (the product literature for the applicable drug should be consulted for complete prescribing information). It is supplied in several sizes. It is packaged individually in (double) blister, sterilized by means of Gamma Irradiation, and is for single use only.

2. Indications  
Topical hemostat for use as an adjunct to hemostasis by tamponade effect, in particular where control of capillary, venous, and arterio-bleeding, by pressure, ligature, and other conventional procedures, is either ineffective or impractical.

3. Contraindications  
• Hypersensitivity to porcine products.  
• GELITA-SPON® RAPID should not be used in closure of skin incisions as it may interfere with healing of the skin edges. This is due to mechanical interposition of gelatin and not to intrinsic interference with wound healing.  
• The product should not be used without antibiotics in infected wounds.

4. Side effects  
• There have been no reported adverse reactions for product used correctly according to these Instructions for Use and when over-booking is not applied.

5. Directions for use  
Sterile technique should be applied when using GELITA-SPON® RAPID.

The sponge is cut into the desired size. The sponge must be applied to a bleeding area under light pressure until the bleeding stops. When bleeding is controlled, the material can be left in situ; however excess should be removed!

Depending on the method of use and the operation site, the GELITA-SPON® RAPID is totally absorbed in less than 4 weeks. When applied to bleeding nasal, rectal or vaginal mucosa, it liquefies within 2 to 5 days.

Important Notice!  
GELITA-SPON® RAPID is compressed and therefore the gelatin sponge expands with the absorption of fluids. In radical cavities; laminectomy procedures, around or in proximity to foramina in bone; areas of bony spine; the spinal cord; the optic nerve and chiasm; and/or closed sinus spaces with presence of bone. GELITA-SPON® RAPID should be removed when bleeding has stopped, if possible. Failure to do so might result in unintended pressure on neighbouring structures which may cause pain for the patient or could create the potential for nerve damage.

While most reports have been in connection with laminectomy, reports of paralysis have also been received in connection with other procedures.

By absorbing fluid GELITA-SPON® RAPID may expand and impinge on neighbouring structures. Therefore care should be taken to avoid over-packing, particularly within bony canals, as it may lead to interference with normal function and may eventually lead to compression neurosis or surrounding tissue.

In cases of postoperative infection, re-operation may be necessary to remove infected material and allow drainage.

Cell saver  
The use of GELITA-SPON® RAPID with a cell saver is not contraindicated. However, make sure to use proper technique when using GELITA-SPON® RAPID in conjunction with a cell saver. As with other hemostatic agents, GELITA-SPON® RAPID should not be aspirated into autologous blood salvage circuits. The technique currently used to ensure that GELITA-SPON® RAPID is not aspirated into such devices is as follows:

1. Prior to initiating the use of GELITA-SPON® RAPID, remove blood salvaging suction tips from the surgical wound and exchange for a wall suction device.

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3. Gently irrigate excess GELITA-SPON® RAPID with your preferred irrigation solution, using the wall suction to remove all loosened matrix. All material not removed by gentle irrigation/flushing is incorporated into the clot.

6. General storage and handling  
GELITA-SPON® RAPID should be stored dry in its sealed protective outer packaging at room temperature avoiding contact with direct sunlight.

Remove the product from the sterile package using sterile technique. Prior to opening, the integrity of the sterile barrier should be confirmed. Discard the product if the packaging is damaged.

Any unused contents should be discarded.

The packaging is subject to and conforms to European & International legislation and other applicable standards. The packaging protects the product from external influences and guarantees its sterility during storage.

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1. Prior to initiating the use of GELITA-SPON® RAPID, remove blood salvaging suction tips from the surgical wound and exchange for a wall suction device.

2. Apply GELITA-SPON® RAPID according to the IFU until hemostasis is satisfactory.

3. Gently irrigate excess GELITA-SPON® RAPID with your preferred irrigation solution, using the wall suction to remove all loosened matrix. All material not removed by gentle irrigation/flushing is incorporated into the clot.

6. General storage and handling

GELITA-SPON® RAPID should be stored dry in its sealed protective outer packaging at room temperature avoiding contact with direct sunlight.

Remove the product from the sterile package using sterile technique. Prior to opening, the integrity of the sterile barrier should be confirmed. Discard the product if the packaging is damaged.

Any unused contents should be discarded.

The packaging is subject to and conforms to European & International legislation and other applicable standards. The packaging protects the product from external influences and guarantees its sterility during storage.

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Die Verpackung ist subject to and conforms to European & International legislation and other applicable standards. The packaging protects the product from external influences and guarantees its sterility during storage.

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