GELITA® EASYTIP S / GELITA® EASYTIP XL

Sterile Flexible Applicators for use with GELITA-CEL® CA-POWDER/ GELITA® ENT X-SPRAY

1. Product description

GELITA® EASYTIP S and GELITA® EASYTIP XL are shapable and disposable applicators, to be used as an inserting system for GELITA-CEL® CA-POWDER and GELITA® ENT X-SPRAY, a sprayable Oxidized Cellulose Powder Hemostat for use as an adjunct to hemostasis in particular where control of capillary, venous, and arteriolar bleeding, by pressure, ligation, and other conventional procedures, is either ineffective or impractical.

Please see the instructions for use for GELITA-CEL® CA-POWDER or GELITA® ENT X-SPRAY for complete information on the absorbable hemostat.

Due to the flexibility of the tube, GELITA® EASYTIP S and GELITA® EASYTIP XL can be curved into the desired shape and will remain in this position, which makes it easy to apply GELITA-CEL® CA-POWDER / GELITA® ENT X-SPRAY also "around the corner".

The GELITA® EASYTIP S has a tube length of 12 cm and is for instance suitable for the application of the Hemostat in ENT procedures like Tonsillectomies.

The GELITA® EASYTIP XL has a tube length of 40 cm and is for instance suitable for the application of the Hemostat in Laparoscopic procedures.

The Applicators are packaged individually in a double pouch, sterilized by ethylene oxide, and are for single use only.

GELITA-CEL® CA-POWDER or GELITA® ENT X-SPRAY are available separately.

2. Indications


GELITA® EASYTIP S and GELITA® EASYTIP XL are intended and designed for the application of GELITA-CEL® CA-POWDER or GELITA® X-SPRAY onto surgical wounds in different medical indications, especially in areas that are difficult to access. Please see the instructions for use for GELITA-CEL® CA-POWDER or GELITA® ENT X-SPRAY for complete information on the absorbable Hemostat.

3. Directions for use

Assembling procedure:

- Sterile technique should be applied.
- Prior to opening of the double pouch, visually inspect the package containing the GELITA® EASYTIP S / EASYTIP XL for any sign of damage. Discard the product if the packaging is damaged or broken.
- Remove the GELITA® EASYTIP S / EASYTIP XL Applicator from the packaging.
- Remove the Harmonica Container with GELITA-CEL® CA-POWDER or GELITA® ENT X-SPRAY from its packaging, applying sterile technique, and then take off its protective cap.
- Firmly press the Harmonica Container containing the Hemostat into the vent of the GELITA® EASYTIP S / EASYTIP XL handle. The GELITA® EASYTIP S / EASYTIP XL Applicator are now ready for use.

- The GELITA® EASYTIP S / EASYTIP XL Applicator should be assembled as shown below.

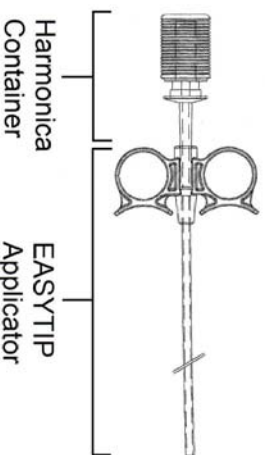Clinical Application:

- Prepare the application site as outlined in the instructions for use for GELITA-CEL® CA-POWDER or GELITA® ENT X-SPRAY.
- Position the GELITA® EASYTIP S / EASYTIP XL Applicator tip with the attached hemostat flask at the side of hemorrhage within the wound, as close as possible to the bleeding area, and apply a thin layer of the powder uniformly to the bleeding site by repeatedly compressing the plastic harmonica container until hemostasis is achieved. After hemostasis is achieved, excess GELITA-CEL® CA-POWDER or GELITA® ENT X-SPRAY should be removed by irrigation and/ or aspiration.
- **Please see the instructions for use for GELITA-CEL® CA-POWDER or GELITA® ENT X-SPRAY for complete information on the absorbable hemostat.**

- Avoid direct contact of the applicator tip with the blood at the surgical site to minimize the potential for clogging up the nozzle during GELITA-CEL® CA-POWDER / GELITA® ENT X-SPRAY application.
- In the event the GELITA® EASYTIP S / EASYTIP XL gets clogged, the applicator can be easily cropped by using a sharp scissor, avoiding crushing of the tip. Avoid cutting the tip over the operation field due to the possibility that parts of the tube may drop into the wound.

4. Contraindications

- Do not use GELITA-CEL® CA-POWDER or GELITA® ENT X-SPRAY in patients with known allergies/hypersensitivity to oxidized cellulose products.
- GELITA-CEL® CA-POWDER or GELITA® ENT X-SPRAY should not be used in bleeding from large arteries, as this may result in embolization of the blood vessel. Such embolization has been associated with severe adverse effects, including fever, duodenal and pancreatic infarct, and embolization of lower

- extremity vessels, pulmonary embolization, splenic abscess, necrosis, astenxis, and death.
- GELITA-CEL® CA-POWDER or GELITA® ENT X-SPRAY should not be used in conjunction with methyl methacrylate adhesives, for example in orthopaedic surgery, because their presence may reduce the adhesive strength of the bonding agent to the bone.
- GELITA-CEL® CA-POWDER shall not be used for injection into bladder or ureteral lumen
- GELITA-CEL® CA-POWDER should not be used for controlling postpartum bleeding or menorrhagia.

5. General Warnings

GELITA-CEL® CA-POWDER or GELITA® X-SPRAY is not intended as a substitute for careful surgery and the proper use of sutures and ligatures or other conventional procedures for hemostasis.

Avoid using the product in infected areas. Embedding the product in a contaminated wound without drainage may lead to complications and should be avoided.

GELITA® EASYTIP S / EASYTIP XL Applicators are supplied as sterile products and cannot be re-sterilized. Unused, open contents should be discarded.

Reusing single-use devices may lead to potentially serious health problems for the patient and malfunction of the product.

6. Precautions

Do not use GELITA® EASYTIP / EASYTIP XL Applicator if the product packaging has been opened or damaged. To prevent product contamination prior to application, always follow aseptic techniques.

The safety and efficacy of the use of GELITA® EASYTIP S / EASYTIP XL with other sprayable hemostatic agents than GELITA-CEL® CA-POWDER or GELITA® ENT X-SPRAY has not been evaluated, and is therefore not recommended.

This product should only be used by surgeons who are familiar with surgical techniques of this type. The surgeon bears the responsibility for the application.

7. General storage and handling

GELITA® EASYTIP S / EASYTIP XL should be stored dry at 0 to 25 °C in its sealed outer protective packaging. Keep away from direct sunlight. Remove the product from the sterile package using sterile technique. Prior to opening, the integrity of the sterile barrier should be

confirmed. Discard the product if the package has been opened or damaged.

Any unused contents must be discarded.

The packaging is subject to and conforms to European and international legislation and other applicable standards. The packaging protects the product from external influences and guarantees its sterility during storage and transport, under normal use.

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GELITA®
 MEDICAL
 Complete Care! Completely Safe!

Explanation of symbols:

	Manufacturer		Store dry
	Date of manufacture		Temperature limitation 0°C - 25°C
	Expiry Date		Single use
	Batch code		Consult instructions for use
	Order number		Sterilized using Ethylene oxide
	Do not re-sterilize		Do not use if the packaging is damaged
	Keep away from sunlight		Latex free

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Angiokard Med. Spritzguss & Entwicklungstechnik GmbH
GELITA MEDICAL GmbH